



The **evolution** in the field of permanent progressive hair removal

Mantis research brings you **TRISOM**, the first laser device featuring **three wavelengths able to work together** and **simultaneously** at three different depths, guaranteeing triple effectiveness.

The synergy of the three wavelengths ensures **TRISOM** is able to work on the broadest and most varied skin and hair types, thanks to the ability of the **ALEXANDRITE**, **DIODE** and **ND:YAG** wavelengths to work simultaneously on the hair's anatomy: the **Hair Bulge**, **Bulb** and **Papilla**.

ND:YAG


sebaceous

gland

hair

bulge

DIODE

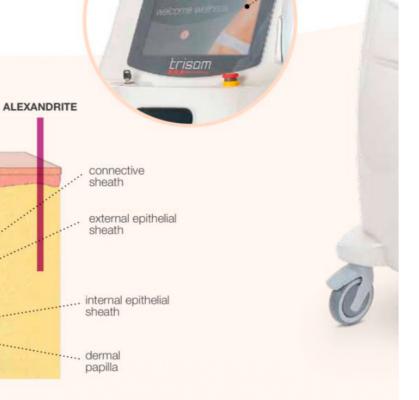





It works at three different depths:

The diode stack is composed of 6 bars with

3 DIFFERENT WAVELENGTHS

that work simultaneously.

TRISOM works at three different depths:

760 nm

(the Alexandrite wavelength)

808 nm

(the typical Diode wavelength)

1064 nm

(the Neodymium-doped YAG wavelength)

Multiwave Technology

- **Diode:** forever used in hair removal, it works deeply, reaching both the Bulge and Bulb. It "moderately" absorbs the melanin and is therefore efficient and safe on all phototypes.
- Alexandrite: it works on a broad range of hair types and colours, thanks to its ability to absorb the energy of the melanin chromophore.

Its "superficial" wavelength (the shortest of the three) aims to act on the Bulge. It is perfect for treating the broadest range of hair/skin combinations, especially light skin and thin hair.

ND:YAG: this is the wavelength with the deepest action, reaching as far as the hair papilla. Its low melanin absorption makes it ideal for treating dark, tanned skin types.



Trisom Medical Laser acts in total respect of the surrounding tissues

TRISOM efficiency, thanks to the liquid optic fibre, is **90% higher** with respect to other lasers. The uniform distribution of energy improves penetration in the follicle area, while the concentration of the light allows improved performance.

New Cooling System

Fibra Cool HP handpiece cooling is extremely efficient thanks to the exclusive Peltier cell system, a technology that guarantees constant cooling for high comfort levels, even in the most sensitive areas of the body. The cooling system uses distilled water.



The software electronically controls the temperature and measures the water flow, thus making it possible to prevent the most common faults that tend to occur in less evolved laser systems.



The thermal capacity of the laser source is enhanced by the ACS - Active Cooling System, guaranteeing virtually unlimited treatment times and increasing the thermal capacity of the entire laser system.











Remote technical assistance

For all its devices, Mantis guarantees remote technical support through an internet connection, and therefore continuous and efficient assistance.

Our results are your success stories!

Clinically tested efficiency for the **face** and **body**. After a few sessions, it is clear that:

- · hair growth is slowed
- the overall amount of hair is reduced









TECHNICAL FEATURES







Laser

Trisom

Monitor

10.4" LCD colour touch screen

Wavelength

760 - 808 - 1064 nm

Laser spot

Ø15 mm

Nominal application distance

16 mm

Energy density

3 ~ 110J

Pulse frequency

1 ~ 12 Hz

Pulse duration

5-300 ms

Beam divergence

(a) 0.5

Electrical power model 230V/50Hz

AC 230V ~ 50 Hz

Absorption 230V/50Hz

900 VA

Electrical power model 110V/60Hz

AC 110V ~ 60 HZ

Absorption 110V/60Hz

1200 VA

Electrical specifications

Class: I TYPE: BF

Measurements

60 x 60 x 130 cm

Net weight

60 Kg



Mantis Srl

Via Artigianale Croce, 13 42035 Castelnovo ne' Monti (RE) - Italy Tel +39 0522 612080 Fax +39 0522 729796 info@mantisitalia.it











RENO1